


STAYS SAFE


VISOR

StaySafe Visor Assembly Instructions


1. Unpeel the backing and stick the foam piece to the centre of the visor 5mm from the top. Use the small crease lines as a guide.

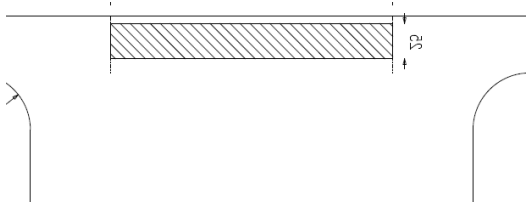

2. Adjust the headband to fit your head
 - Put the thin side of the headband through the two slits on the wider side, in towards the back of the visor and then out towards you

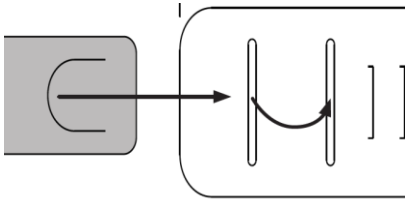

- Then fit the D-shaped tab into a suitable slot for your head size

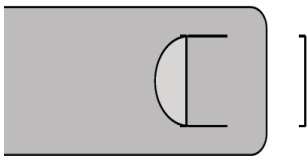

The visor is now ready to wear.

To re-order or see the other products in our range visit our website www.staysafevisor.co.uk

For enquiries over 5000 units please email sales@1st-packaging.co.uk

